

Butterfly iQ Educational View Guidance (Supported Devices Only)

The Educational View Guidance tools provide users with a visual indication of the quality of the image while scanning with the Butterfly iQ. The Educational View Guidance tools support the following views: Cardiac Apical 4 Chamber, Parasternal Long Axis and Parasternal Short Axis and Lung A-Lines/B-Lines view. As you scan the subject, the tool provides real-time feedback. It indicates the proportion of experts who would assess the anatomical view as measurable.



CAUTION!

The Educational View Guidance tools are for Educational Use Only.


Compatibility

Educational View Guidance is supported on all Butterfly iQ Compatible iOS devices:

Mobile support: iOS version 12.0 or newer

Accessing Educational View Guidance

The Educational View Guidance tools can be accessed in the Cardiac or Lung Preset while scanning in B-Mode.

Tap **Actions**  located at the bottom right corner.

You have the option to select the following tools from the Educational View Guidance section:

- Cardiac preset: Apical 4 Chamber, Parasternal Long Axis and Parasternal Short Axis
- Lung preset: A-Lines/B-Lines

Turning off Educational View Guidance

The Educational View Guidance tools can be turned off by tapping **X** on the tool while:

- Scanning in B-mode
- Or in the Actions menu